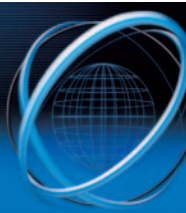
ARCO® 3000 / 2000 / 1000

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ARCO® 3000

Electro-surgical unit with max. functions: monopolar, gas, bipolar. Mainly used in: general surgery, gastroenterology.

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ARCO® 2000

Electro-surgical unit with monopolar and gas functions. Mainly used in: gastroenterology.

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ARCO® 1000

Electro-surgical unit for monopolar gascoagulation for argon enhanced hemostases.

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Standardly built on a trolley. Trolley standardmäßig enthalten.

ARCO®

Extremely user-friendly

The latest generation of electro-surgical equipment incorporates the most advanced user-friendly technology including a self-adjusting cutting system and the unique Söring Endoflow® System. The control panel is clearly defined with colour coded coagulation and cutting fields. The equipment is present to five power and gas flow output levels most frequently used by surgeons, but the digital display facilitates rapid fine adjustment of those present levels to the operating surgeon's preferences. All ARCO® units are trolley mounted for easy manoeuvrability and can additionally be mounted on rock systems.

Additionally all units (except for ARCO®1000) offer the possibility to store up to 10 individually adjusted programs permanently.

Improved safety features

Endoflow®

The Söring Endoflow® Systems automatically selects the optimum power and gas output level for the handpiece or probe connected, thus eliminating the risk of error or oversight.

Safety equipment

The traffic-light alarm system continually monitors the placement of the patient electrode and its contact to the patient. Visual and audio alarms alert the user to wrongly connected, misplaced or damaged patient-electrodes, and the system switches off automatically before critical parameters are reached.

A permanent software control of all components completes the safety features.

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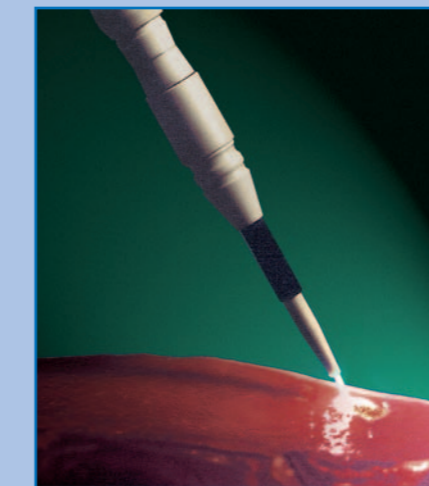
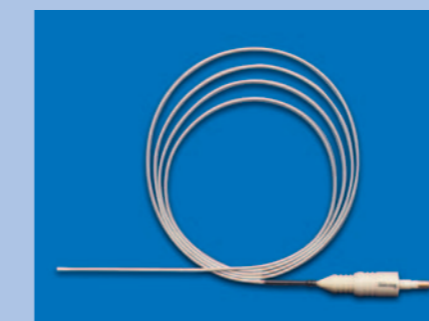
Argon-Plasma Coagulation. Argon-Plasma Koagulation.

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ARCO®

Argon coagulation

A beam of pure ionized argon gas induces rapid coagulation with minimal tissue damage.

Other benefits include:

- Reduced operating time and costs,
- Enhanced healing due to exceptionally fine yet flexible and durable eschar,
- Minimized blood loss and replacement needs,
- Good visibility, reduced smoke production or burning odour.

Monopolar cutting

MBC® units automatically select the optimum cutting intensity, independent from electrode and tissue resistance.

Pure-Cut: Standard HF-cut without additional coagulation effects.

TUR: Achieves excellent results in fatty tissue or under water cuts, e.g. in urology (trans-urethral resection) or orthopedics (arthroscopy).

Blend: This mode allows for cutting and simultaneous coagulation of the raw surface. There are up to 18 degrees (9 x Blend + 9 x Power Blend).

Power Blend: This cutting effect offers a dry cut. Due to the reduced cutting speed an enhanced coagulation effect is generated.

Pulse: In this mode a pulsed current greatly enhances safety when cutting in visually obscured areas, as in polypectomy or similar procedures.

TUR-Pulse: Enhanced power intense version of the Pulse-Cut for safe cuts in low power range, e.g. laparoscopic operations in urology or orthopedics.

Monopolar coagulation

Three modes are available:

Soft: Conventional contact-coagulation in the low-power range (dehydration).

Force: Forced contact-coagulation in the increased power range – 10! effects free adjustable (denaturation).

Spray: Spray-coagulation without tissue contact (vaporization).

The special Hi-Force function offers perfect coagulation even under water – without undesired sparks that could create a cutting effect.

Bipolar cutting and coagulation

Additional safety is achieved via this bipolar technique. RF-current is conducted directly between the tips of tweezers. BipoPulse mode for nearly bleeding free bipolar cuts. Unique compatibility with instruments from different manufacturers (Tissue Link & Everest Medical).

Functions can be activated/de-activated by footswitch or by automic function. A remote-footswitch with switching function between "monopolar" and "bipolar" functions offers the user an simplified handling.

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